

USER INSTRUCTIONS

TP_SURF – ear-plugs that you can adjust exactly to the shape of your ear. Each set contains two sizes of sealing gels and two sizes of flaps. All elements can be freely combined to achieve the best user conveniences.

Method of use:

1. Insert TP_SURF ear-plugs into your ear. Make sure the ear-plugs are correctly inserted with the flap up [see photograph].
2. If necessary, change the size of the ear-plug sealing gels or flaps for the best sealing and protection effects.

Important: Each ear canal is a different shape, therefore, please try all the combinations of the ear-plug sealing gels and flap sizes that you have in the set. A close fit is essential for proper sealing, protection, and maximum comfort.


How to resize TP_SURF ear-plug sealing gels.

1. Grasp the flap with one hand and pull the ear-plug sealing gel with the other hand.
2. Place a new sealing gel on the tip with the narrower part of the core (blue part).
3. Put the sealing gel on the stem by pressing it gently with your thumb.
4. Your ear-plug is ready to use.

Important: Each ear canal is a different shape, therefore, please try all the combinations of the ear-plug sealing gel and flap sizes that you have in the set. A close fit is essential for proper sealing, protection, and maximum comfort.



Each wing has the size marked on it: M or L.

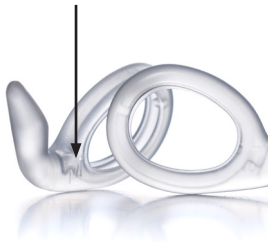

How to exchange TP_SURF flaps.

1. Hold the TP_SURF ear-plug with one hand. Place your index finger on the ear-plug sealing gel and your thumb on the wider blue core element of the TP_SURF (blue part inside the flap), then grasp the loose part of the flap with two fingers of the other hand.
2. Gently lift up the loose part of the flap.
3. Remove the flap from the wider part of the ear-plug core (blue part).
4. Put the new flap on the protruding element of the wider part of the ear-plug core (blue part).
5. While holding it, gently pull the remaining part of the flap over the wider blue element of the ear-plug core.
6. Make sure the flap and pin of the ear-plug form a smooth surface. Finished!

Note!

Each wing has the size marked on it: M or L.

How to care for TP_SURF

For the sake of the proper functioning of TP_SURF, the following hygiene rules should be observed:

1. After each use, wash all TP_SURF parts in clean running water and soap. Dry thoroughly.
2. When you are not using the TP_SURF ear-plugs, keep them away from direct sunlight.
3. Store TP_SURF ear-plugs in the enclosed protective packaging in a cool, dry place.

Note!

Wash TP_SURF ear-plugs regularly to remove any salty water. Salt crystals can permanently damage the functioning of the product.

1. flaps
2. sealing gels
3. the core
4. the leash

